

USER MANUAL

4-point Cane

Strong and functional 4-point Cane specially designed to the bariatric user.

4-point Cane can be used outdoors and indoors in institutions, care centers and in private homes. 4-point Cane is intended users weighing up to 270 kg.







Table of content

1. Safety	3
2. Symbols	
3. General	
4. Use	
5. Settings	5
6. Long-term storage	6
8. Disposal	6
7. Cleaning and maintenance	6
9. Warranty	7
10. Specifications	
11. Ordering	8

1. Safety



WARNING

Do not use the 4-point Cane without prior thorough instructions by trained staff or by persons who have received instructions in the use of the cane. Improper operation may cause injury to the user.

If the 4-point Cane is used by persons weighing more than the maximum user weight, the risk of damaging the 4-point Cane increases.

Carefully study these instructions before using the 4-point Cane. Pay attention to Safety Instructions indicated by a warning triangle \triangle .

Cobi Rehab accepts no responsibility for damage caused to the product nor for personal injury caused by improper installation, assembly or use of the 4-point Cane.

Contact Cobi Rehab immediately in case of damaged or missing parts. Never use your own spare parts.



SAFETY INSTRUCTIONS

The 4-point Cane can break or be damaged if used by a user weighing over 270 kg.

If used by a person weighing more than 270 kg, there is a risk that the 4-point Cane can sustain damage.

4-point Cane may not be used before the dual button height adjustment is pressed fully in place, otherwise there is risk that the inner tube might loosen, causing the user a risk of falling.


Always remember to disinfect 4-point Cane when transferred to a new user. This is done to avoid cross-contamination and infections.

Failure to inspect the dual button may cause malfunction.

Lack of maintenance can damage the product, shorten the life, or affect the performance of the 4-point Cane.

2. Symbols

The following symbols are used in this user manual and/or on the product label.			
\triangle	Caution!		
C€	Declaration of conformity, Medical Device Regulation, EU 2017/745		
REF	Cobi Catalogue number		
SN	Serial number		
	Date of manufacture		
	Manufacturer		
MD	Medical device		
۴	Body weight		
[]i	Consult instructions for use		
CE-Label on the product is located on the shaft. See example of label below.			

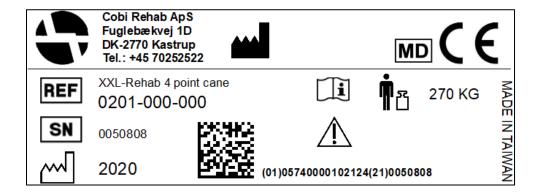

3. General

This user guide contains important information about the use of 4-point Cane and the target group for the cane. It also includes a wide range of specifications on 4-point Cane.

The most recent version of this user guide can be found on our website. Check if the version you are reading now is the most recent.

If you need a user manual that is written in a larger font size, the guide is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

You can always contact Cobi Rehab on + 45 7025 2522 or cobi@cobi.dk if you want more information about 4-point Cane.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

The UDI number of the crutch appears on the label, which is located on the side of the shaft.

Product safety notices and product recalls can be viewed on the website.

https://www.cobirehab.com/kunde-hos-cobi/vigtig-produktinformation/

4. Use

4-point Cane is a cane specially designed to provide support and safety while walking for users weighing up to 270 kg.

4-point Cane is used at hospitals, care centers and in private homes. 4-point Cane should only be used on level ground and accessible terrain, indoors as well as outdoors.

⚠ If used by persons weighing more than 270 kg, there is a risk that the 4-point Cane can sustain damage.

Always consult a therapist before assigning a 4-point Cane to a bariatric user.

Contraindications

The user of 4-point Cane must have physical and cognitive resources to operate the cane in a safe manner.

5. Settings

4-point Cane is supplied with a dual button height adjustment, a rubber handle, and a wrist strap.

Before use

The hand support is height adjustable by pressing the dual button height adjustment.

▲ 4-point Cane may not be used before the dual button height adjustment is pressed fully in place, otherwise there is risk that the inner tube might loosen, causing the user a risk of falling.

6. Long-term storage

4-point Cane should be stored at temperatures between -10°- 50°C.

7. Cleaning and maintenance

4-point Cane can be cleaned with water and ordinary detergent.

Disinfection

4-point Cane can be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm.

Always remember to disinfect 4-point Cane when transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

The dual button height adjustment should be regularly inspected and if necessary, replaced.

 \triangle Failure to inspect the dual button may cause malfunction.

8. Disposal

Products that can no longer be used are to be disposed of in a sustainable way. Please contact the relevant authorities for information on proper disposal.

Recycling

Parts of 4-point Cane can be reused. Contact Cobi Rehab for more information.

9. Warranty

When delivered, the 4-point Cane is free from defects. Should there be any defects or damage to the 4-point Cane, contrary to all inspections, please contact Cobi Rehab immediately.

The 4-point Cane is covered by a 1-year warranty from the date of purchase. The invoice serves as warranty certificate.

The warranty does not apply to:

- products whose serial number, batch number etc. have been removed or considerably damaged
- products that have been repaired by unauthorized staff
- normal wear and tear, for example cleaning, adjustment, lubrication and update of equipment or parts.

9.1 Unintended event

Report any incidents related to the use of 4-point Cane to Cobi Rehab.

The warranty is void if Cobi Rehab assesses that the 4-point Cane has been used for purposes other than its intended use, for example, if its weight capacity is exceeded.

In case of incorrect use, both service and spare parts will be invoiced.

The warranty described above is the only valid warranty.

Cobi Rehab may not be held liable for damages of any kind.

⚠ Lack of maintenance can damage the product, shorten the life, or affect the performance of the 4-point Cane.

9.2 After the expiry of expected service life

After the expected service life, it is important that an overall assessment of the product is made before continued use.

After the expected service life, Cobi Rehab cannot guarantee the suitability and safety of the product, as Cobi Rehab has no control over how the product is used and wear and tear.

The overall assessment of the product must be carried out by qualified professionals, and as a

minimum, consideration must be given to how the product has been used, the condition of the product and its components.

Cobi Rehab can always make such an assessment.

10. Specifications

Maximum user weight	270 kg		
Height adjustable	71-93 cm		
Weight	1.5 kg		
Base size	15 x 22 cm		
Reflex in grip	Yes		
Material	Chrome molybdenum steel/SAE41-30		
Grip	TPR - Thermoplastic rubber		
ISO - classification	ISO 12 03 16 - 03		
Expected product service life	7 years		
Risk class	1		
UDI-DI	5740000102858		
Basic UDI	5740000100010MV		
The surface temperature of the 4-point Cane can increase or decrease when exposed to external sources of heat or cold.			

11. Ordering

Model	Height adjustable	Item number
4-point Cane	71-93 cm	0201-000-000





MANUFACTURER

Cobi Rehab

Fuglebækvej 1D DK-2770 Kastrup Denmark +45 7025 2522

cobirehab.com