

PRODUCT SHEET Cobi Cruise Power bariatric comfort wheelchair

Cobi Cruise Power comfort wheelchair. The most powerful wheelchair on the market for use in the rehabilitation of bariatric users.

Cobi Cruise Power comfort wheelchair is used in care centres and hospitals, in home care and in private homes for bariatric users in need of a comfort wheelchair with a pressure-relieving seat.



- Used in all phases of rehabilitation, from the bedridden user to the mobile user.
- Cobi Cruise Power is equipped with electric propulsion, tilt-in-space and recline functions. All functions are activated via a controller and operated by both user and caregiver.
- Armrests that can swivel back or be "pushed down" for e.g., transfers or social activities at a table.
- Disinfectable and washable in laundry room at e.g., a depot or a hospital.
- Fits through most doorways.



Scan or click on the code and find the product at the AssistDATA.

Cobi Rehab and Cobi Rehab suppliers comply with the UN conventions on child labour and forced labour as well as ILO 029, C132 and C182.

MD

Cobi Cruise Power comfort wheelchair

Cobi Cruise Power is based on the challenges of a daily routine to improve quality of life for each bariatric user. Therefore, Cobi Cruise Power is tested by Danish bariatric users, thus contributing with valuable suggestions for improvement.

Driving a Cobi Cruise Power gives freedom. Freedom to move. From bedridden to independent. Cruise also means safety for both user and caregiver.

Functions like electric propulsion, push bar, electric tilt-in-space, and recline considerably reduce the risk of physically overloading the caregiver, while still allowing maximum comfort, mobility, and social life for the user.

General comments

This product sheet contains important information about the use of Cobi Cruise Power and the target group for the wheelchair. It also includes a wide range of specifications on the Cobi Cruise Power.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet that is written in a larger font size, the product sheet is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

You can always contact Cobi Rehab on telephone +45 7025 2522 or e-mail cobi@cobi.dk if you want

Application

The Cobi Cruise Power comfort wheelchair is designed for indoor use, e.g., in hospitals, institutions and in care homes. The wheelchair can also be used in private homes. The wheelchair has a width that allows it to pass through most doorways.

The wheelchair is intended for bariatric users up to 325 kg whose ability to stand and walk is limited.

The intended operator of the wheelchair is a health professional, such as a therapist, nurse, or a social and health worker. The intended operator could also be the bariatric user with trunk stability Cobi Cruise Power is designed entirely by Danish engineers and Danish occupational therapists and tested at House of Practice and Innovation at University College Copenhagen.

Cobi Cruise Power is produced in the EU and successfully tested with a 500 kg/1100 lb. load.

more information about Cobi Cruise Power comfort wheelchair.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

The UDI number of the wheelchair appears on the label, which is located on the side of the frame.

Product safety notices and product recalls can be viewed on the website.

and the ability to drive, control and break the wheelchair.

The wheelchair can be used in ambient temperatures of 0°-40°C and a humidity of 20-80%.

The wheelchair can also be used outdoors but is primarily for indoor use.

Construction

Cobi Cruise Power is constructed from S355, high-tensile steel. The rectangular steel profiles of the chair form the basis for its base frame. The base frame is made up of two mirror-inverted parts (a right and a left part) connected by a centre module. This structure allows the sides to be reused if the width needs changing.

The seat frame of the wheelchair consists of 30x30x2 mm high-tensile steel profiles. In addition, the seat frame has a 2 mm steel seat plate attached to it, which helps to distribute the pressure in the best possible way.

The back of the wheelchair consists of a hightensile steel profile. The profile is positioned centrally on the back of the wheelchair. The angle of the profile means that the back supports both apple-shaped and the pear-shaped users. Three back plates are mounted on the profile. The back plates are curved to hug and support the user from buttocks to neck.

Pear-shaped users with voluminous backsides are ensured good back support, as the design of the "spine" of the wheelchair and its lower back plate provide space for the backside.

The upper two back plates ensure that appleshaped users enjoy stable and comfortable back support.

The back cushion is divided into nine modules. Each module consists of two parts. A firm foam layer and layer with a mixture of polystyrene (Krøyer) beads and latex foam.

The foam layer faces the back of the wheelchair while the bead and foam layer are the one sat on by the user. The bead and foam mixture are modular and helps to increase the comfort of the wheelchair and its pressure relieving effect.

The contents of in each cushion module can be adjusted. This means that, according to professional assessment, the therapist or caregiver can add or remove beads and latex foam. If necessary, a wedge cushion can also be inserted in one or more modules. The cushion contents are adjusted via the nine zippers located on the back of the cushion.

The modular cushion follows the shape of the back plates and, thus, ensures correct support regardless of user body shape and needs.

The modular cushion can be finely adjusted using a patented FitGo system known from ski boots and work shoes. The FitGo system is located on the back plates.




Curved plates hug and support the user from buttocks to neck.



The back cushion is made up of nine modules sewn into one large cushion.



Each module consists of a firm foam layer and a layer of beads and latex foam.

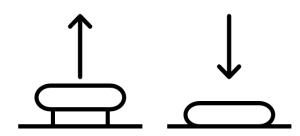

The FitGo system comprises an adjusting knob. The button is activated when pressed. If the button is to be loosened, pull it upwards and the cord pull is then released.

The seat cushion

The base of the seat cushion consists of hard foam. The core of the seat cushion consists of memory foam specially selected for the target group.

The top is made of soft foam. The seat cushion is upholstered with a flame retardant incontinence cover.

Push bar

The push bar is flexible and adjustable, using a click system. Major adjustments can be made with tools. The bar has several grip points, depending on whether the chair is to be pushed or manoeuvred.

Armrests

Extra-long armrests (615 mm) ensure stability and a good grip when the user moves from a standing to a sitting position or vice versa.

The armrests can swivel back, making it easier to move the user from e.g., the chair to the bed. The armrests can also be pushed down making it possible to move the wheelchair close to the edge of a table, allowing the user to participate in social activities there.

The armrests are padded for extra comfort and "cup-shaped" thus reducing the risk of pressure ulcers. Each armrest can withstand a maximum load of up to 200 kg.



The seat cushion is made up of three different types of foam.



Swiveling arm rests allow positioning near table.

Leg rests

The leg rests of the wheelchair are an option. The leg rests each support up to 200 kg and are adjustable in both height and position. They also have a swivel function that allows easier access to and from the wheelchair.

The angle of the foot plates, the height/angle of the leg rest and the angle of the calf plates can be adjusted without tools.

The flexibility of the leg rests means that the user's legs can always be supported. This benefit, inter alia, users with a significant tissue on their inner thighs.

The leg rests may be removed without the use of tools without affecting stability.



Leg rests each supporting up to 200 kg.



The leg rests can be dismantled without using tools.

Neck support

The wheelchair neck support is an option. The wheelchair is supplied with a universal neck support bracket, allowing for the installation of different types of neck support.

Wheels

Cobi Cruise Power is equipped with two 12,5" massive tyres in grey nylon rubber that does not mark floors. The tyres are mounted on aluminium rims.

Brakes

The brake of the Cobi Cruise Power is integrated into the motor. This means that when the joystick is disengaged, the wheelchair stops.

Actuators

The Cobi Cruise Power has three actuators. Two seat actuators and a back actuator. The two seat actuators control the tilt-in-space function. The seat actuators are installed under the seat.

The back actuator controls the recline function. The back actuator is also installed under the seat.

Anti-tip

Cobi Cruise Power is installed with anti-tip that can be adjusted to suit the terrain.

Charger

The Cobi Cruise Power must be charged until the battery indicator reaches 100%. A full charge can take up to six hours.

During charging, the functions of the wheelchair cannot be used.

The control box

The control box is installed on the base plate.

The electronics are IPX4-classified. This means that the wheelchair and its components can withstand a light rainfall.

The electronics of the wheelchair does not disturb the operation of units in the surrounding environment which emit electromagnetic fields, e.g., alarm systems in shops, automated doors, etc.



A universal neck support bracket permits the installation of several types of neck supports.



Two actuators (marked blue) control the tilt-in-space function. One actuator (red) controls the recline function.



The electronics of the wheelchair do not affect electromagnetic fields emitted by e.g., laptops and electricity generators.



Electric tilt-in-space and recline

Negative tilt helps the independent user in safely getting in and out of the wheelchair. Negative tilt means that the front of the seat cushion is lowered. This reduces the access height which, in turn, makes getting in and out easier for the independent user.

The function negative tilt is activated using the controller.

Joystick

The supplied joystick allows care staff and user to manoeuvre the wheelchair and to adjust the angle of the seat and back (tilt-in-space -4-13° and 90-120° recline).

The joystick is installed on the supplied swivel bracket and can be positioned on either the inside

Recline provides the apple-shaped user with supported and comfortable seating. When the back of the wheelchair is tilted backwards, the pelvis of the user is automatically opened. This provides improved respiration for apple-shaped users.

The recline function is controlled using the same controller as for the electric tilt-in-space function.

or the outside of the armrest.

An extra joystick controlling the same functions can be installed on the push bar. This joystick can also determine who controls the wheelchair. In this way, the user's option of controlling it can be activated or deactivated.

The joystick has:

- An on/off button
- A button with a horn
- A display showing the battery capacity, time, speed, and travel mode
- two buttons marked Mode and Profile
- a twin button for controlling speed.

Display

The battery capacity is displayed at the top. Avoid travelling long distances on low battery capacity.

At the bottom, a number between 1 and 3 is displayed. The number indicates the travel mode, where 1 is manoeuvring, 2 is indoor and 3 is outdoor. Each travel mode has five speed increments. The speed in kilometres is displayed above the five speed increments. The speed is displayed in numbers and illustrated with a speedometer.

Profile switches between travel modes 1 to 3.

Press *Mode* to switch between travel and seat and back adjustment. Use the up-down movement of the joystick to control the tilt-in-space function. Switch between the tilt-in-space and recline function by moving the joystick right or left.

When the back and seat are tilted into an unstable function, speed options are restricted. This is indicated by an orange or red turtle being displayed.





Travel

The control box for travel is installed underneath the seat plate. The wheelchair can reach a speed of up to 6 km/h.

The maximum range is approx. 9 km at full load. The theoretical manoeuvring distance is 3.9 km at full load.

Maintenance and cleaning

The Cobi Cruise Power is easy to clean. If the seat needs disinfecting, remove, and disinfect it using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

Clean the frame, seat and arm rests of the wheelchair using water and a regular cleaning agent.

The wheelchair can traverse edges and doorsteps of up to 50 mm. Cobi Cruise Power is also built to be able to drive up and down hills and ramps of a 10° incline. For the user, it means freedom to get around both indoor and outdoor.

Cobi Cruise Power comfort wheelchair can be disinfected or washed in a laundry room at e.g., a depot or a hospital.

The controller can be disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

Transport and storage

When the wheelchair is not in use, store in a dry place at a temperature between -10° - 50° C.

The Cobi Cruise Power is suitable for land and/or air transport (when not accommodating a user). The Cobi Cruise Power cannot be folded as it has a fixed frame. To avoid injuries to e.g., the back, we advise against carrying the 120 kg wheelchair. The wheelchair can be lifted by the frame if it is to be transported (without a user) in a motorised vehicle.

Long-term storage

When storing the Cobi Cruise Power for longer periods of time (3+ months), store the wheelchair at 0°-30° degrees and in a humidity not exceeding 75%.

We recommend charging the battery to 40–60% prior to storage.

When putting the wheelchair into use again after long-term storage, connect it to the supply and charge it for at least two hours. Keep an eye on the battery capacity as the wheelchair will discharge over time.

Specifications

Max. user weight	325 kg		
Risk class			
Basic UDI DI	5740000100023N6		
Wheelchair class	A		
Expected product service life	Minimum 7 years		
Crash-tested	No, but tests will be completed in early 2021		
Seat depth	480-600 mm, at 30 mm intervals		
Back angle	90°- 120°, continuously variable		
Seat angle	-4 - 13°		
Height from front of seat to ground (Height stated without cushion. Add 100 mm if cushion	included) 337-551 mm		
Height of back rest	613 mm		
Max. length excl. footrests / incl. footrests	Approx. 1246 mm - 1773 mm		
Min. length excl. footrests / incl. footrests	Approx. 987 mm - 1054 mm		
Seat width 550-800 mm, at 50 m	nm intervals. 100 mm between 700 and 800 mm		
Max. total width	676-926 mm		
Max. total height	1115 mm		
Max. weight of wheelchair	120 kg		
Max. height of curb which the wheelchair can safely trav	verse 50 mm		
Max. downhill inclination	10°		
Max. uphill inclination	10°		
Max. sideways inclination	10°		
Massive tyres	Nylon tyres with PU core		
Speed at electric propulsion	6 km/h		
Theoretical continuous distance at max weight capacity	Approx. 9 km		
Theoretical manoeuvring distance	3.9 km		
Frame tubing (material)	Type-S355, high-tensile steel		
Seat cushion filling CMHR 50250 (base), Visco	MHR 50250 (base), Visco 45060 (core), CMHR 4038 (top), flame-retardant		
Back cushion filling CM 2740 (base), Cut	latex and EPS-beads (the front), flame-retardant		
Back and seat cushion Covered in in	Covered in incontinence cover (G469 black), flame-retardant		
Arm rest cushion Foam (CMHR4250 and RF	n (CMHR4250 and RF5060), cover durable imitation leather (Temskay Leather), flame-retardant		
Cushion in foot and Foam (CMHR4038), co lower leg support	ver durable imitation leather (Temskay Leather), flame-retardant		

Push bar	Type-S355, high-tensile steel		
Wheel forks	Aluminium		
Footrest tubing	Steel		
Foot plate	Type-S355, high-tensile steel		
Turning radius (Pivot)	100-120 cm		
Foldable	No		
Battery	Lithium 50 Ah 24V		
Three actuators	LINAK LA23		
Control box	LINAK		
Controller	Curitss-Wright R-net PM120		
Electric wheelchair, including components	DS/EN 12184: 2014, according to ISO 7176		
IP-code	IPX4		
Risk assessment	EN/ISO 14971: 2012		
General requirements and test methods	EN 12182:2012		
Accessories and spare parts	Part number		
Leg rest R	0115-999-006		
Leg rest L	0115-999-007		
Cobi Cruise Power IV-holder with	0115-999-090		

Cobi Cruise Power IV-holder with installation bracket(s)	0115-999-090	
Neck support	Contact us for the choice of neck supp	

Ordering					
Version	Seat width mm	Seat depth mm	Max. total width mm	Item number	
Electric version with joystick	550	480-600	670,6	0116-055-000	
Electric version with joystick	600	480-600	720,6	0116-060-000	
Electric version with joystick	650	480-600	770,6	0116-065-000	
Electric version with joystick	700	480-600	820,6	0116-070-000	
Electric version with joystick	800	480-600	920,6	0116-800-000	





MANUFACTURER

Cobi Rehab Fuglebækvej 1D DK-2770 Kastrup Danmark +45 7025 2522

cobirehab.com