

JAY® Ion™ Cushion



Owner's Manual

IMPORTANT CONSUMER INFORMATION.

NOTICE: This manual contains important instructions that must be passed on to the user of this product.

Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.







	03			
ENGLISH		3		
GENERAL WARNINGS		3		
JAY ION CUSHION		4		
JAY ION COMPONENTS		4		
FITTING A PRESSURE DISTRIBUTING CUSHION				
EASY MAINTENANCE & CLEANING				
REASSEMBLING THE JAY	Y ION CUSHION			
AFTER CLEANING		5		
JAY ION CUSHION WARF	RANTY	5		
APPROVED CLEANING P	ROCEDURES FOR			
JAY CUSHIONS		6		

NOTE: Check all parts for shipping damage. In case of damage DO NOT use. Contact Carrier/Sunrise for further instructions.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY lon Cushion is appropriate.

Cushions should only be installed by an authorized Sunrise Medical supplier.

GENERAL WARNINGS



DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified

clinician or supplier - otherwise injury or damage may occur.

⚠ WARNING

Hook and Loop velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the velcro.

riangle warning

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

⚠ WARNING

Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.

The JAY Ion cushion is designed to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores. The JAY Ion cushion is not a substitute for good skin care including; proper diet, cleanliness, and regular pressure relief.

CAUTION – AVOID SHARP OBJECTS OR EXPOSURE TO EXCESSIVE HEAT OR OPEN FLAME.

MARNING

OBSTRUCTIONS: DO NOT place any obstructions between the user and the cushion because it will reduce product effectiveness.

riangle warning

COVER ORIENTATION: Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

MAXIMUM USER WEIGHT: 340 mm to 520 mm (14" to 21") width = 136 kg (300 lbs.)

540 mm to 600 mm (22" to 24") width = 226 kg (500lbs.)


JAY ION CUSHION

The JAY Ion is designed with the benefits of skin protection and positioning in a lightweight, comfortable and low-maintenance design. The JAY Ion incorporates an microclimatic outer cover, a moisture resistant inner cover and a dual layer foam base.

The JAY Ion helps to provide pressure and shear reduction which may help reduce the incidence of decubitus ulcers. The cushion is intended to be comfortable and does not restrict movement, allowing the user to change positions on the cushion.

The maximum user weight for this product is 136 kg (300 lbs.) for 340 mm to 520 mm (14" to 21") widths and 226 kg (500lbs.) for 540 mm to 600 mm (22" to 24") widths.

JAY ION COMPONENTS

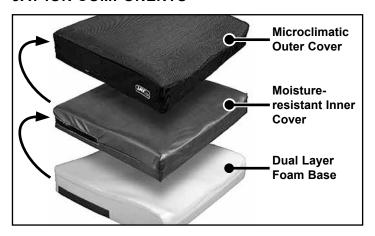

FITTING A PRESSURE DISTRIBUTING CUSHION

Select a cushion to match your hip width when in the seated position. The cushion length should extend to within 1" - 2" (25mm - 50mm) from the back of the knees. Place the cushion to the back of the wheelchair with the pelvic loading area toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered in the rear of the cushion. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the JAY Adjustable Solid Seat.

Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

EASY MAINTENANCE & CLEANING

Regular cleaning and maintenance may help extend the life of your cushion.

During cleaning, component inspection is recommended. Check the cover for tears and excessive wear and or any other abnormalities.

Check the foam base to ensure foam consistency.

To clean cover

- Remove the outer cover from the foam base and zip closed.
- Machine wash in warm water and drip or tumble dry on low heat.

No ironing required.

NOTE: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE: If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.

To clean foam base

Remove the cover from the foam base. Wipe lightly with a damp cloth. Do not use soap. Do not submerge in water. Wipe off with a clean cloth. Let it completely air-dry before reassembly.

NOTE: Avoid submersion of the foam base in water.

REASSEMBLING THE JAY ION CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in "Easy maintenance and Cleaning" then reassemble as follows.

Insert the foam base into the inner cover with white label down and zipper to rear of cushion. Then insert base plus inner cover into the outer cover. Make sure white inner cover label is down and zipper is to the rear. Ensure that both zippers are at the rear of the foam base (the part number tag should be to the rear of the foam base) where the pelvis is typically located.

JAY ION CUSHION WARRANTY

Every JAY Ion cushion base is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable outer and inner covers are guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of nonconforming goods sold.

Approved Cleaning Procedures for JAY Cushions											
Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance		
Outer Cover (Microclimatic and Stretch)											
(1). Wet Wash 70°C	. (. (. (. (. (
70	70	70	70	70	70	70	70	70	70		
Outer Cover (Incontinent)											
(2). Wet Wash 60°C	,										
60	60	60	60	60	60	60	60	60	60		
Inner Cover											
(3). Spray and Wipe disinfection 60 or Wet Wash 60°C	NA	NA	NA	NA	NA	60	NA	NA	60		
Cushion core (foam bas	e)			<u> </u>				<u> </u>			
(4). Steam autoclave											
up to 105 °C (1)	×	×	×	×	×				×		
Spray and wipe disinfection						×	×	×	×		
(5). Immersion disinfection	THE STATE OF THE S			×	×	×	×	×	×		
Cushion elements											
Roho Air element	NA			NA	NA	NA	NA	NA			
Positioning elements	NA			NA	NA		NA	NA			
Fluid pad	NA			NA	NA		NA				

- 1.70°C is the maximum recommended temperature.
- 2.60°C is the maximum recommended temperature.
- 3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.
- 4. An autoclave is a self locking machine that sterilizes with steam under pressure.
- 5. At low temperature (< 30°C)

The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute: http://www.rki.de/EN/Home/homepage_node.html



Sunrise Medical S.r.I. Via Riva, 20 – Montale 29122 Piacenza

Italia

Tel.: +39 0523 573111 Fax: +39 0523 570060 www.SunriseMedical.it

Sunrise Medical AG Erlenauweg 17 CH-3110 Münsingen Schweiz/Suisse/Svizzera Fon +41 (0)31 958 3838 Fax +41 (0)31 958 3848 www.SunriseMedical.ch

Sunrise Medical AS Delitoppen 3 1540 Vestby Norge Telefon: +47 66 96 38 00 post@sunrisemedical.no www.SunriseMedical.no

Sunrise Medical AB Neongatan 5 431 53 Mölndal Sweden Tel.: +46 (0)31 748 37 00 post@sunrisemedical.se www.SunriseMedical.se

MEDICCO s.r.o. H – Park, Heršpická 1013/11d, 639 00 Brno Czech Republic Tel.: (+420) 547 250 955 Fax: (+420) 547 250 956 www.medicco.cz info@medicco.cz Bezplatná linka 800 900 809


Sunrise Medical Aps Mårkærvej 5-9 2630 Taastrup Denmark +45 70 22 43 49 info@sunrisemedical.dk www.Sunrisemedical.dk

Sunrise Medical Australia 11 Daniel Street Wetherill Park NSW 2164 Australia Ph: +61 2 9678 6600

Email: enquiries@sunrisemedical.com.au www.SunriseMedical.com.au

Sunrise Medical North American Headquarters 2842 Business Park Avenue Fresno, CA, 93727, USA (800) 333-4000 (800) 300-7502 www.SunriseMedical.com

REP

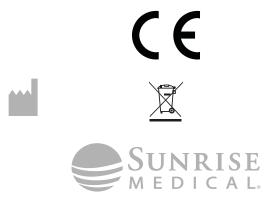

OM_JAY ION_Rev.D_EU_EN_2020-11-23

Sunrise Medical GmbH Kahlbachring 2-4 69254 Malsch/Heidelberg Deutschland

Tel.: +49 (0) 7253/980-0 Fax: +49 (0) 7253/980-222 www.SunriseMedical.de

Sunrise Medical Thorns Road Brierley Hill West Midlands DY5 2LD England Phone: 0845 605 66 88 Fax: 0845 605 66 89 www.SunriseMedical.co.uk

Sunrise Medical S.L. Polígono Bakiola, 41 48498 Arrankudiaga – Vizcaya España Tel.: +34 (0) 902142434 Fax: +34 (0) 946481575 www.SunriseMedical.es

Sunrise Medical Poland Sp. z o.o. ul. Elektronowa 6, 94-103 Łódź Polska Telefon: + 48 42 275 83 38 Fax: +48 42 209 35 23 E-mail: pl@sunrisemedical.de www.Sunrise-Medical.pl

Sunrise Medical B.V. Groningenhaven 18-20 3433 PE NIEUWEGEIN The Netherlands T: +31 (0)30 - 60 82 100 F: +31 (0)30 - 60 55 880 E: info@sunrisemedical.nl www.SunriseMedical.nl

Sunrise Medical HCM B.V. Vossenbeemd 104 5705 CL Helmond The Netherlands T: +31 (0)492 593 888 E: customerservice@sunrisemedical.nl www.SunriseMedical.nl www.SunriseMedical.eu (International)

Sunrise Medical S.A.S ZAC de la Vrillonnerie 17 Rue Mickaël Faraday 37170 Chambray-Lès-Tours Tel: + 33 (0) 247554400 Fax: +30 (0) 247554403 www.sunrisemedical.fr