

USER MANUAL

Bedside commode/Shower Chair Clean

Strong and functional bedside commode/shower chair intended for bariatric users.

Clean is used in care homes, institutions, and hospitals. User weight up to 325 kg. Available in two sizes, 61 cm, and 71 cm width.







Table of content

1. Safety	3
2. Symbols	
3. General comments	
4. Use	5
5. Setting	5
6. Long-term storage	6
7. Cleaning and maintenance	6
8. Disposal	6
9. Warranty	7
10. Specifications	
11. Ordering	8

1. Safety



WARNING

Do not use the Bedside Commode Clean without thorough instructions from trained staff or from persons who have received instruction in the use of the Bedside Commode Clean. Improper operation may injure the user.

Using the Bedside Commode Clean for people weighing more than the maximum user weight of 325 kg increases the risk of damaging the Bedside Commode Clean.

Read this user manual carefully before using the Bedside Commode Clean. Pay particular attention to Safety Instructions marked \triangle .

Cobi Rehab assumes no responsibility for any damage to the product nor for personal injury caused by incorrect installation, incorrect mounting, or incorrect use of the Bedside Commode Clean.

Immediately contact Cobi Rehab if parts are damaged or missing. Never use your own spare parts.

Cobi Rehab cannot be held responsible for errors or accidents occurring after repairs are made without written permission from Cobi Rehab.



SAFETY INSTRUCTIONS

Bedside Commode Clean may tip or break if the max. weight limit is exceeded. Maximum user weight 325 kg.

Do not use Bedside Commode Clean in pools or swimming pools.


The surface temperature can rise when Bedside Commode Clean is exposed to external heat sources e.g., sunlight.

Bedside Commode Clean may only be used on a flat, hard floor with a maximum inclination of 3°. Larger slope increases the risk of tipping.

Bedside Commode Clean must be assembled correctly before use. See section Setting.

2. Symbols

The following symbols are used in this user manual and/or on the product label.			
\triangle	Caution!		
CE	Declaration of conformity, Medical Device Regulation, EU 2017/745		
REF	Cobi Catalogue number		
SN	Serial number		
	Year of manufacture		
	Manufacturer		
MD	Medical device		
r L	Max. user weight		
[]i	Consult the manual		
CE label located at the frame.			
	See an example of an CE-label below.		

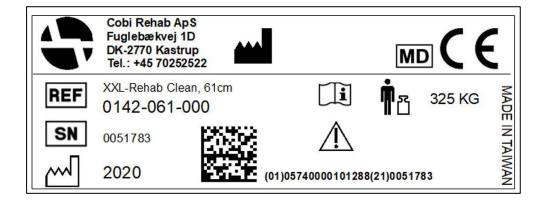

3. General comments

This user manual contains important information about the use of Bedside Commode Clean and the target group for Bedside Commode Clean. It also includes a wide range of specifications on the Bedside Commode Clean.

The most recent version of this user manual can be found on our website. Check if the version you are reading now is the most recent.

If you need a user manual that is written in a larger font size, the product sheet is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

You can always contact Cobi Rehab at + 45 7025 2522 or cobi@cobi.dk if you want more information about the Bedside Commode Clean.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

The UDI number of Bedside Commode Clean appears on the label, which is located on the side of the frame.

Product safety notices and product recalls can be viewed on the website.

4. Use

Bedside Commode Clean is intended for users weighing up to 325 kg during lavatory use and in shower situations.

Bedside Commode Clean is used in hospitals, care centers and institutions.

⚠ Bedside Commode Clean may tip or break if the max. weight limit is exceeded.

⚠ Do not use Bedside Commode Clean in pools or swimming pools.

Always consult a therapist before assigning Bedside Commode Clean to a user.

Contraindications

Monitor users with cognitive impairment while using the Bedside Commode Clean.

5. Setting

Assembly of Bedside Commode Clean

Bedside Commode Clean consists of two parts, the chair frame, and the backrest.

1. Put a washer on each bolt. Fastened the backrest to the chair frame by inserting the four bolts and washers into holes, two on each side. Tighten them well with the accompanying hexagon allen key.

Adjustment of the seat depth and backrest

1. The seat depth can be adjusted into three positions. Choose two holes (front, middle or back) on the backrest for the desired seat depth. See the instruction above.

Adjustment of the height settings of the leg

1. The height of the legs can be adjusted into six positions by loosening and tightening the four bolts already mounted on the legs. Use both hexagon allen keys when adjusting.

Tools to use

1. Two 5 mm hexagon allen keys, four hex sockets allen key bolts, four washers.

During use

The backrest can be adjusted in three different depths. Adjusting the seat depth does not affect the stability of the chair.

6. Long-term storage

When the Bedside Commode Clean is not in use, store it in a dry place and between -10 - 40°C.

7. Cleaning and maintenance

The Bedside Commode Clean frame and seat is cleaned using water and a regular cleaning agent.

Disinfection

Bedside Commode Clean can be disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

△ Always remember to disinfect Bedside Commode Clean when transferred to a new user.

This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

Maintenance

Bolts and washers should be checked regularly and replaced if necessary.

⚠ Failure to check bolts and washers as instructed may cause malfunction.

8. Disposal

Dispose of products that are no longer in use in an environmentally safe manner.

Contact your local authority or Cobi Rehab for further information on disposal.

Recycling

Contact Cobi Rehab for more information on recycling of parts of the Bedside Commode Clean.

9. Warranty

Cobi Rehab warrants that the Bedside Commode Clean is without defects on delivery. If, against all expectations, the Bedside Commode Clean is subject to deficiencies or defects on receipt, please contact Cobi Rehab immediately.

The warranty period for the Bedside Commode Clean is one year from the date of purchase and covers manufacturing defects or defect on receipt. The invoice is your certificate of warranty.

The warranty does not apply to:

- products on which the serial number, batch number or the like have been removed or significantly damaged.
- products which have been repaired by unauthorised personnel.

9.1 Unintended events

Report any incidents related to the use of the Bedside Commode Clean to Cobi Rehab.

The warranty does not cover:

- wearing parts.

This warranty does not apply when Cobi Rehab assesses that the Bedside Commode Clean has been improperly operated.

For damage caused by improper operation, an invoice will be issued for both the service and the spare parts used.

🗥 Lack of maintenance can damage the product, shorten the life, or affect the performance of the Bedside Commode Clean.

9.2 After expiry of the expected service life

After the expected service life, it is important that an overall assessment of the product is made before continued use.

After the expected service life, Cobi Rehab cannot guarantee the suitability and safety of the product, as Cobi Rehab has no control over how the product is used and wear and tear.

The overall assessment of the product must be carried out by qualified professionals, and as a

minimum, consideration must be given to how the product has been used, the condition of the product and its components.

Cobi Rehab can always make such an assessment.

10. Specifications

Maximum user weight	325 kg		
Seat width	610/710 mm		
Seat depth adjustable	400-520 mm		
Seat height adjustable	440-590 mm		
Distance between arm rests	650/750 mm		
Total width	730/830 mm		
Total height	790-940 mm		
Width toilet opening	Ø 240 mm, 350 mm long		
Seat/back/arm rests	Polyurethane		
Frame	Powder-lacquered chromium molybdenum steel/SAE41-30		
Weight (61 cm/71 cm)	14/15 kg		
ISO classification	09 33 07 03		
Expected product service life	7 years		
Risk class	1		
UDI-DI	5740000101288/5740000101295		
Basic UDI	5740000100003MY		
The surface temperature can rise when the Bedside Commode Clean is exposed to external heat sources e.g., sunlight.			

11. Ordering

Model	Item number
Bariatric Bedside Commode Clean, 61 cm	0142-061-000
Bariatric Bedside Commode Clean, 71 cm	0142-071-000
Seat for Bedside Commode Clean, 61 cm	0150-999-022
Seat for Bedside Commode Clean, 71 cm	0150-999-023
Bedpan with cover	0140-999-001
Ferrules, rubber, 1 piece	0140-999-112





MANUFACTURER

Cobi Rehab Fuglebækvej 1D DK-2770 Kastrup Denmark +45 7025 2522

cobirehab.com