


USER MANUAL Shower Commode

Strong and wide Shower Commode with space under the backrest for the tissue in the gluteal region. Adjustable footrests so users with a lot of tissue on the inside of the thighs can sit with legs abducted.

Shower Commode is used in care homes, institutions, and hospitals. User weight up to 325 kg.

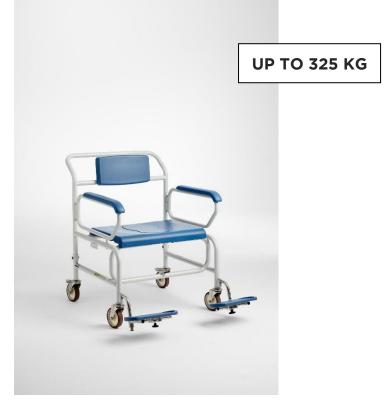

- Brakes on two out of four wheels.
- Optional extra-wide butterfly arm rests.
- Available in two sizes, 61 cm og 71 cm.
- Shower Commode ensures a better working environment for the caregiver and a comfortable and safe bathing experience for the user.
- Extra-long arm rests.
- Several types of leg rests.



Scan the QR-code and find the product on the Danish AssistData.

Cobi Rehab and Cobi Rehab suppliers comply with the UN conventions on child labour and forced labour as well as ILO 029, C132 and C182.

MD

Table of content

. Safety	3
2. Symbols	4
. General comments	
l. Use	5
5. Settings	6
5. Cleaning and maintenance	7
'. Transport and storage	7
B. Disposal	8
). Warranty	8
0. Specifications	9
1. Accessories and spare parts	10

1. Safety



WARNING

Do not use the Shower Commode without thorough instructions from trained staff or from persons who have received instruction in the use of the Shower Commode. Improper operation may injure the user.

Using the Shower Commode for people weighing more than the maximum user weight of 325 kg increases the risk of damaging the Shower Commode.

Read this user manual carefully before using the Shower Commode. Pay particular attention to Safety Instructions marked \triangle .

Cobi Rehab assumes no responsibility for any damage to the product nor for personal injury caused by incorrect installation, incorrect mounting, or incorrect use of the Shower Commode.

Immediately contact Cobi Rehab if parts are damaged or missing. Never use your own spare parts.

Cobi Rehab cannot be held responsible for errors or accidents occurring after repairs are made without written permission from Cobi Rehab.

SAFETY INSTRUCTIONS

In this user manual, the symbol ${\rm \Delta}$ indicates a safety instruction. Follow safety instructions carefully.

Always lift users not able to stand and walk in and out of the Shower Commode.

The maximum user weight of the Shower Commode is 325 kg.

The surface temperature can rise when the Shower Commode is exposed to external heat sources e.g., sunlight.

Shower Commode may only be used on a flat, hard floor with a maximum inclination of 3 °. Larger slope increases the risk of tipping.

Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

Remember to lock the wheels with the wheel locks when the user is seated on Shower Commode, except when it is used for transport.


Failure to inspect bolts and lock nuts may result in malfunction.

Lack of maintenance can damage the product, shorten the life, or affect the performance of Shower Commode.

Always remember to disinfect Shower Commode when transferred to a new user to avoid cross-contamination and infections.

2. Symbols

The following symbols are used in this user manual and/or on the product label.			
	Caution!		
CE	Declaration of conformity, Medical Device Regulation, EU 2017/745		
REF	Cobi Catalogue number		
SN	Serial number		
M	Year of manufacture		
***	Manufacturer		
MD	Medical device		
۳ß	Max. user weight		
Ĩi	Consult the manual		
CE label located at the frame.			
See an example of an CE-label below.			

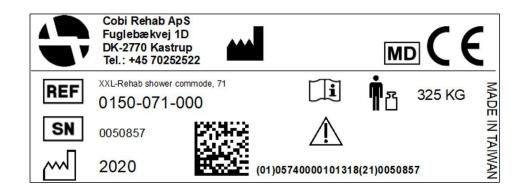

3. General comments

This user manual contains important information about the use of Shower Commode and the target group for Shower Commode. It also includes a wide range of specifications on the Shower Commode.

The most recent version of this user manual can be found on our website. Check if the version you are reading now is the most recent.

If you need a user manual that is written in a larger font size, the product sheet is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

You can always contact Cobi Rehab at + 45 7025 2522 or cobi@cobi.dk if you want more information about the Shower Commode.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

The UDI number of Shower Commode appears on the label, which is located on the side of the frame.

Product safety notices and product recalls can be viewed on the website.

4. Use

Shower Commode is designed to help the bariatric user during transfers to and from the bathroom, in the shower, and during lavatory use.

Shower Commode is intended for indoor use in e.g., hospitals, care centers and institutions.

Shower Commode is developed for users up to 325 kg with reduced physical capacity.

The maximum user weight of the Shower Commode is 325 kg.

A Shower Commode is not intended for traversing obstacles such as doorsteps, etc. The

shower commode should only be used on a firm surface to avoid the risk of tipping. The intended operator of the Shower Commode is a health professional, such as a therapist, nurse, or social and healthcare worker.

Shower Commode can be used in environments with a temperature range of 0-40°C and a humidity interval of 20-80%.

Contraindications

Monitor users with cognitive impairment while using the Shower Commode.

5. Settings

Before use

The armrests must be properly installed in the sockets on the side of the Shower Commode.

Make sure that the seat is correctly mounted on the frame of Shower Commode.

To prevent the risk of the user falling off the side of the Shower Commode, never use it without properly installing the armrests first.



Shower Commode with butterfly arm rests.

During use

Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

Pinch hazards 🛆

- Installing the armrests while the user is seated on Shower Commode may present a pinch hazard between the armrests and the side of the commode.

- There is a risk of pinching fingers and other parts of the body between the Shower Commode's seat frame and the toilet or bedpan. Therefore, be particularly careful when placing Shower Commode over the toilet or inserting a bedpan beneath the seat.

Both wheels must be locked with the wheel locks when the user is seated on Shower Commode, except when it is used for transport.



Seat entirely with cover.



The bedpan is easy to insert and empty.

6. Cleaning and maintenance

The Shower Commode frame, seat and arm rests can be cleaned using water and a regular cleaning agent.

Disinfection

The seat can be removed and disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

▲ Always remember to disinfect Shower Commode when transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

Shower Commode is washable in a cabinet washer.

A Failure to inspect bolts and lock nuts may result in malfunction.

7. Transport and storage

When the Shower Commode is not in use, store it in a dry place and between -10 - 40 °C.

The Shower Commode is suitable for land and/or air transport (without the user). The Shower Commode cannot be folded as the frame is fixed.

To avoid injuries to e.g., the back, care staff should refrain from carrying the 27 kg shower commode.

If it is to be transported, the Shower Commode can be lifted by the frame, without the user being seated on it, into a motor vehicle.

7.1 Long-term storage

When storing the Shower Commode for a longer period (more than three months), store it in 0-40°C and in humidity not exceeding 80%.



Swing-away arm rests.

8. Disposal

Dispose of products that are no longer in use in an environmentally safe manner.

Contact your local authority or Cobi Rehab for further information on disposal.

9. Warranty

Cobi Rehab warrants that the Shower Commode is without defects on delivery. If, against all expectations, the Shower Commode is subject to deficiencies or defects on receipt, please contact Cobi Rehab immediately.

The warranty period for the Shower Commode is one year from the date of purchase and covers manufacturing defects or defect on receipt. The invoice is your certificate of warranty.

The warranty does not apply to:

- products on which the serial number, batch number or the like have been removed or significantly damaged.

Recycling

Contact Cobi Rehab for more information on recycling of parts of the Shower Commode.

- products which have been repaired by unauthorised personnel.

The warranty does not cover:

- wearing parts.

This warranty does not apply when Cobi Rehab assesses that the Shower Commode has been improperly operated.

For damage caused by improper operation, an invoice will be issued for both the service and the spare parts used.

Lack of maintenance can damage the product, shorten the life, or affect the performance of the Shower Commode.

9.1 Unintended event

Report any incidents related to the use of the Shower Commode to Cobi Rehab.

9.2 After expiry of the expected service life

After the expected service life, it is important that an overall assessment of the product is made before continued use.

After the expected service life, Cobi Rehab cannot guarantee the suitability and safety of the product, as Cobi Rehab has no control over how the product is used and wear and tear.

The overall assessment of the product must be carried out by qualified professionals, and as a minimum, consideration must be given to how the product has been used, the condition of the product and its components.

Cobi Rehab can always make such an assessment.

10. Specifications

Max. user weight	325 kg
Risk class	
Basic UDI DI	5740000100004N2
Expected product service life	7 years
Total length	1170 mm
Total width with standard arm rests (without footrests	s) 780/880 mm
Total width with butterfly arm rests (without footrests) 870/970 mm
Seat width with standard arm rests	610/710 mm
Seat width with butterfly arm rests	750/850 mm
Seat depth	560 mm
Seat height	550 mm
Width between arm rests	60 mm
Width toilet opening	Ø 240 mm, 350 mm long
Distance from ground to under the seat	445 mm
Distance from ground to push bar	1000 mm
Distance from seat to the upper edge of the back	450 mm
Distance from seat to footrests	400/420/440/460/480 mm
Total weight (model 61 cm/71 cm)	27 kg
Swing away arm rests	Yes
Footrests swing away, height adjustable	Yes
Footrest material	PU foam
Individual wheel locks	Yes, 2 pieces
Turning radius	1580 mm
Frame	Powder lacquered chromium molybdenum steel
Wheels	Ø125 mm, massive, 4 pieces
Seat/back/arm rests	Polyurethane
IP classification	IPX4
ISO classification	09 12 03 21
Risk assessment	EN/ISO 14971:2012
General requirements and test methods	EN 12182

11. Accessories and spare parts

Accessories	Varenummer
Butterfly arm rest right (adds 7 cm width between the arm rest)	0150-999-012R
Butterfly arm rest left (adds 7 cm width between the arm rest)	0150-999-012L
Seat for Shower Commode Wheeler, 61 cm	0150-999-022
Seat for Shower Commode Wheeler, 71 cm	0150-999-023
Bedpan with cover	0140-999-001
Footrest standard shower commode right	0156-999-090H
Footrest standard shower commode left	0156-999-090V
Lateral leg rest for shower commode right	0156-999-030
Lateral leg rest for shower commode left	0156-999-031





MANUFACTURER

Cobi Rehab Fuglebækvej 1D DK-2770 Kastrup Denmark +45 7025 2522

cobirehab.com